

# deconex® OPHTHALMICS H

**Ready to use cleaner for the daily care  
of ophthalmic lenses**

For manual application



## Application

deconex® OPTHALMICS H is a ready to use cleaner for the daily cleaning and care of ophthalmic lenses.

## Properties

deconex® OPTHALMICS H removes dust, fingerprints and other deposits due to daily glass wearing. deconex® OPTHALMICS H does not leave any residues and has a pleasant odour.

## Ingredients

Solvents, wetting agent, scent

## Dosage

Apply deconex® OPTHALMICS H on suitable, scratch-free cloth and wipe glasses.

## Chemical/physical data

Density	concentrate	0.88 g/mL
Appearance	concentrate	clear, colourless
Flash point	concentrate	24 °C

## Availability

Please ask your local representative about current container sizes.

Containers, screw caps and labels are made of recyclable polyethylene.

## Information on use

deconex® OPTHALMICS H contains inflammable solvents. Therefore, avoid storage above 22 °C and protect from direct sun.

## Material compatibility

Suitable for:  
Mineral lenses, plastic lenses

For materials not mentioned please make your own specific compatibility tests or consult Borer Chemie AG.

## Additional information

For information concerning safety at work, storage and waste disposal/effluent, please consult the corresponding safety data sheet.

Take advantage of our vast know-how! Please, contact us for further information regarding your specific application.

# deconex® OPTHALMICS H

**Manufacturer:**

**Borer Chemie AG**

Gewerbestrasse 13, 4528 Zuchwil / Switzerland

Tel +41 32 686 56 00 Fax +41 32 686 56 90

office@borer.ch, www.borer.swiss

All information provided is based on our current knowledge and it does not constitute a legally binding assurance of specific product properties.