

deconex® dosing table for disinfectants and detergent concentrates

Instructions to prepare a ready-to-use solution of a deconex® disinfectant or detergent. The quantity of use solution consists of tap water and the respective disinfectant or detergent concentrate (e.g. to receive 10 litres of a 2% solution, 200 ml of concentrate will have to be added to 9.8 litres of tap water). Follow manufacturer's recommendations and guidelines.

Percentage of ready-to-use solution	0.25 %	0.50 %	0.75 %	1.00 %	1.50 %	2.00 %	3.00 %	4.00 %	5.00 %
Total volume of ready-to-use solution	Required amount of disinfectant or detergent concentrate (ml)								
0.5 litre	1.25	2.5	3.75	5	7.5	10	15	20	25
1 litre	2.5	5	7.5	10	15	20	30	40	50
2 litre	5	10	15	20	30	40	60	80	100
3 litre	7.5	15	22.5	30	45	60	90	120	150
4 litre	10	20	30	40	60	80	120	160	200
5 litre	12.5	25	37.5	50	75	100	150	200	250
6 litre	15	30	45	60	90	120	180	240	300
7 litre	17.5	35	52.5	70	105	140	210	280	350
8 litre	20	40	60	80	120	160	240	320	400
9 litre	22.5	45	67.5	90	135	180	270	360	450
10 litre	25	50	75	100	150	200	300	400	500
15 litre	37.5	75	112.5	150	225	300	450	600	750
20 litre	50	100	150	200	300	400	600	800	1000
25 litre	62.5	125	187.5	250	375	500	750	1000	1250
30 litre	75	150	225	300	450	600	900	1200	1500